# MANUALE D'USO - USER MANUAL

BORGES LABORATORIA, S.P.A. (ITALY) - MEDICAL DEVICES

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**INTENDE USE**  
**WARNING:**  
- use this device only as described in this manual and therefore as an aerosol therapy system, following the indications of your doctor. Any use different from the intended one is to be considered improper and hence dangerous; the manufacturer cannot be held liable for any damage caused by improper, incorrect and/or unauthorised use of the equipment is connected to electrical interferences which do not comply with current safety regulations.  
2. Keep this manual for future reference.  
3. Do not operate the unit in presence of any anaesthetic mixture inflammable with oxygen or nitrogen protoxide.  
4. The power plug of the equipment can be affected by electromagnetic interferences which exceed the limits indicated by the European standards in force. In case of failure and/or malfunction, read the "POSSIBLE PROBLEMS AND HOW TO SOLVE THEM" section. Do not handle or open the compressor housing.  
5. For replacement of the nebulizer or other parts not authorised by the manufacturer and require the use of original spare parts. The non-observation of the above mentioned indications can compromise the device safety.  
6. Comply with the safety regulations concerning the electrical devices and in particular:  
- use only original accessories and components.  
- never submerge the unit in water;  
- never touch the device, it is not protected against water penetration;  
- never touch the unit with wet or moist hands;  
- do not leave the unit exposed to the weather elements;  
- place the unit on a stable and horizontal surface during its operation;  
- the use of this device by children and disabled requires always the close supervision by an adult with full mental faculties;  
- do not pull the power cord or the device itself to unplugh it from the power socket;  
- the power plug must be unplugged from the wall outlet when the device is in use.  
8. Before plugging in the device, make sure that the electrical rating, shown on the rating plate on the bottom of the unit, corresponds to the mains rating.  
9. In case of the power plug provided with the device does not fit your wall socket, address to qualified personnel for the replacement of the plug with a suitable one.  
10. Do not leave the equipment so long in use, making sure the main supply switch is turned on, that it is not used for long periods of time.  
11. The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or things, for which the manufacturer cannot be held responsible.  
12. The power cord of this device cannot be replaced by the user. In case of a power cord damage, address to a technical service center authorized by the manufacturer for its replacement.  
13. The power supply cord should always be fully unwound in order to prevent dangerous overheating.  
14. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.  
15. Some parts of the unit are so small that they may be swallowed by children; keep the equipment out from children's reach.  
16. If you decide not to use the device any longer, it is recommended to dispose of it according to the current regulations.  
17. Make sure to:  
- use this device only with medicines prescribed by your doctor;  
- make the treatment only when the necessary recommendation by your doctor depending on the pathology;  
- use the nosepiece accessory only if expressly indicated by your doctor and paying attention to NEVER introduce the bifurcations in the nose, but only using bringing them as close as possible.  
18. Check the original package leaflet for possible contraindications for use with common aerosol therapy systems.  
19. To avoid strangulation and entanglement, keep cable and air tubes out of reach of young children.  
20. Do not position the equipment so that it is difficult to operate the disconnection device.  
21. For greater safety, we recommend you to use the same equipment for more than one person.

**INSTRUCTIONS FOR USE**  
The device must be checked before each use, in order to detect possible functioning anomalies and/or damages due to transport and/or storage.  
During inhalation, sit upright at a table and do not sit in an armchair, in order to avoid compressing your respiratory tract and impairing the treatment effectiveness.  
After unpacking the device, check for visible damages or defects; pay particular attention to cracks in the plastic housing, which may expose electrical components. Check for accessories integrity.  
2. Before using the device, proceed with the cleaning operations as described in the "CLEANING AND MAINTENANCE" section.  
3. Open the nebulizer lid.  
4. Make sure that the medicine conduction cone is properly fitted into the air conduction cone inside the nebulizer (Pic. C).  
5. Put the prescribed quantity of medicine into the nebulizer (Pic. D).  
6. Connect the nebulizer to the power supply, making sure that the power plug is correctly inserted into the wall outlet (Pic. E).  
7. Connect one end of the air tube to the nebulizer (Pic. F) and the other end to the air outlet (Pic. G) on the device.  
8. If you need to use the mask, insert it directly onto the nebulizer (Pic. H).  
9. Plug the device into the wall outlet, making sure the main supply corresponds to the electrical rating of the device.  
10. To start the treatment, set the I/O switch into the "P" position.  
**ATTENTION: This device is for intermittent use: 30 min ON / 30 min OFF.**  
11. When the treatment has been completed, switch the unit off by setting the I/O into the "O" position and disconnect the plug from the wall socket.  
12. Wash the nebulizer and its accessories as described in the "CLEANING AND DISINFECTION OF THE ACCESSORIES" section.  
13. The device requires no calibration.  
**DOUBLE FUNCTION - AEROSOL THERAPY**  
With Family Plus you can choose between two delivery modes: maximum effectiveness or maximum speed.  
**Maximum effectiveness of therapy (mode A)**  
This mode is intended for the treatment of chronic diseases. To make inhalation therapy more effective, turn the valve on the top of the nebulizer to letter "A" (valve closed). Use this mode only with the mouthpiece. Do not use the mask or the nose fork.  
**Maximum speed of therapy (mode B)**  
This mode increases the delivery speed. To make inhalation therapy faster, turn the valve on the top of the nebulizer to letter "B".

**CLEANING AND MAINTENANCE**  
The cleaning of the device must be carried out by using a soft and dry cloth and non-abrasive cleansers.  
**ATTENTION:** During cleaning operations, make sure the internal parts of the unit are not in contact with liquids and that the power plug is not connected.  
**CLEANING AND DISINFECTION OF THE ACCESSORIES**  
Clean and disinfect the cleaning and disinfection instructions of the accessories as they are very important for the device performance and the therapy success.  
**1. After each treatment:**  
Disassemble the nebulizer by turning counterclockwise the top and remove the medicine conduction cone. Wash the components of the disassembled nebulizer with clean water. Wash the mouthpiece and nosepiece by using tap water; dip in boiling water for 5 minutes. Ressemble the nebulizer components and connect it to the air-outlet (Pic.G).  
**2. Do not boil nor autoclave the air tube.**  
It is possible to use cold disinfecting liquids following the manufacturer's instructions.  
**REPLACEMENT OF THE NEBULIZER**  
The nebulizer must be replaced after a long period of inactivity, in case it shows deformations or breakings, or when the nebulizer nozzle is obstructed by dry medicine, dust, etc. Medel recommends to replace the nebulizer after a period between 6 months and 1 year depending on the usage.  
**Use original models only.**  
**REPLACEMENT OF THE AIR FILTER**  
In normal use conditions, the air filter must be replaced approximately after 200 working hours or after each year. Medel recommends to periodically check the air filter. If the air filter shows a grey or brown color, replace it. Extract the filter (Pic. A-5) and replace it with a new one. Do not try to clean the filter. The air filter shall not be recycled or maintained while in use with the apparatus.  
**3. Use original filters only. Do not use the device without filter.**  
**POSSIBLE PROBLEMS AND HOW TO SOLVE THEM**  
Contact an authorized Customer Service Centre in case you need help about the use and the maintenance of the equipment.  
**PROBLEMS:**  
1. The device does not switch on  
2. The device does not nebulize or nebulizes weakly  
**SOLUTIONS:**  
1. Make sure the power plug is firmly fitted to the wall socket;  
2. Make sure that the device has been operating within operating limits indicated in this manual (30 min ON / 30 min OFF).  
3. Make sure that the ends of the air tube are tightly fitted to the main unit and nebulizer;  
4. Verify that the nebulizer is filled with the proper quantity of medicine (max 8 ml). Verify that the nebulizer nozzle is not obstructed.  
Should the device not start working properly again, contact the **Medel CUSTOMER SERVICE**.  
**MAINTENANCE AND REPAIRS:** In case of failure, address to qualified personnel authorized by Medel International S.r.l.. Do not open the device in any case. The unit has no user-serviceable parts within and does not need internal maintenance or lubrication.

**AEROSOL CHARACTERISTICS ACCORDING TO EN 15344-1 ANEXO C**  
**Flow:** 5,1 l/min  
**Aerosol size:** 0,25 ml  
**Maximum filling volume:** 2 ml  
**Maximum filling rate:** 0,089 ml/min  
**Nebulization rate:** 0,37 ml/min (mode B)  
**Maximum pressure:** 2,2 bar  
**Weight:** 1,5 kg  
**Size:** 188 x 0,136 m  
**Noise level:** 52 dBa  
**Expected service life: 1000 hours**  
Class II device as regards protection against electric shocks.  
Nebulizer, mouthpiece and mask are type BF applied parts.  
Device not protected against splashes.  
Device not suitable for use in presence of anaesthetic mixtures inflammable with air, oxygen or nitrogen protoxide.  
Device for intermittent use (30 min ON / 30 min OFF).  
Device not suitable for use in anaesthesia or lung ventilation systems.  
**NOTE:** The technical specifications may change without prior notice.  
**ELECTRIC SAFETY STANDARDS EN 60601-1:** Electromagnetic Compatibility according to EN 60601-1-2. Class IIa Medical Device according to the 93/42/EEC European Directive on Medical Devices.

**ENVIRONMENT CONDITIONS**  
**Storage / Transport:** Temperature: MIN -25 °C - MAX +70 °C  
Humidity: MIN 10% RH - MAX 95% RH  
Atmospheric pressure: 700 hPa - 1060 hPa  
**OPERATION**  
Temperature: MIN +10 °C - MAX +40 °C  
Humidity: MIN 10% RH - MAX 95% RH  
Atmospheric pressure: 700 hPa - 1060 hPa

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